



## Contents

|                                       |    |
|---------------------------------------|----|
| (EN) Instructions for Use.....        | 1  |
| (ES) Instrucciones de uso .....       | 3  |
| (FR) Manuel d'utilisation .....       | 5  |
| (HR) Upute za uporabu.....            | 7  |
| (JA) ユーザーズガイド .....                   | 9  |
| (KK) Пайдалану нұсқаулығы .....       | 11 |
| (KO) 사용 설명서 .....                     | 13 |
| (MK) Инструкции за користење .....    | 15 |
| (PT-BR) Instruções de Uso .....       | 17 |
| (RU) Инструкция по эксплуатации ..... | 19 |
| (SR) Uputstvo za upotrebu .....       | 21 |
| (UK) Інструкції з використання .....  | 23 |
| (ZH-CN) 使用说明 (书) .....                | 25 |
| (ZH-TW) 使用说明 .....                    | 27 |



www.philips.com/healthcare/eIFU

|                                      |  |
|--------------------------------------|--|
| (BG) Указания за употреба .....      |  |
| (CS) Návod k použití.....            |  |
| (DA) Brugerhåndbog.....              |  |
| (DE) Gebrauchsanweisung.....         |  |
| (EL) Οδηγίες χρήσης.....             |  |
| (ET) Kasutusjuhend.....              |  |
| (FI) Käyttöohjeet .....              |  |
| (HU) Használati útmutató .....       |  |
| (IT) Istruzioni d'uso .....          |  |
| (LT) Naudojimo instrukcija .....     |  |
| (LV) Lietošanas instrukcija .....    |  |
| (NL) Gebruiksaanwijzing.....         |  |
| (NO) Bruksanvisning .....            |  |
| (PL) Instrukcja obsługi.....         |  |
| (RO) Instrucțiuni de utilizare ..... |  |
| (SK) Návod na obsluhu .....          |  |
| (SL) Navodila za uporabo .....       |  |
| (SV) Bruksanvisning .....            |  |
| (TR) Kullanım Talimatları.....       |  |

## DESCRIPTION

Philips non-invasive blood pressure (NIBP) cuffs are available to suit a variety of patient applications and are color-coded for easy size selection (see *Select Appropriate Cuff Size*).

## SYMBOL DEFINITIONS

The following are definitions for symbols that might appear on package labels as well as the products described in this document:

|  |   |  |
|--|---|--|
|  | = | Quantity per package                                   |
|  | = | Quantity per box                                       |
|  |   | Proper application of cuff                             |
|  |   | Improper application of cuff                           |
|  |   | Patient limb circumference range (cm)                  |
|  |   | Follow operating instructions                          |
|  |   | Prescription use only                                  |
|  |   | Not manufactured with natural rubber latex             |
|  |   | Not manufactured with polyvinyl chloride (PVC)         |
|  |   | Not manufactured with Di(2-ethylhexyl)phthalate (DEHP) |
|  |   | Caution  |
|  |   | Storage temperature range                              |
|  |   | Storage humidity range                                 |

## INDICATIONS

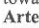
Philips multi-patient and single-patient cuffs are intended for use by, or under the supervision of, a licensed physician or other healthcare provider for the non-invasive measurement of infant, pediatric, and adult human blood pressure.

## WARNINGS

- Turn off auto function or remove cuff from monitor when not on patient.
- When cleaning cuffs, use care to prevent fluid from entering tubing. Use end cap, if provided with cuff. Do not immerse tubing in any fluids.
- Avoid placing cuff on patient in a manner that can lead to a cuff tube becoming kinked (a kinked tube can cause inaccurate readings).
- Single-patient (disposable) cuffs are not intended for use on more than one patient.
- To help to extend the service life of Easy Care cuffs, the cover fabric is treated with an antimicrobial agent that helps to prevent stains and odor caused by bacteria and fungi. Reduction in colonization of microbial growth on the device has not been shown to correlate with a reduction in patient infections. Clinical studies to evaluate reduction in infection have not been performed.

## APPLICATION INSTRUCTIONS

**Note:** Refer also to the illustration on the front cover. Numbers in bold in the procedure that follows correspond to the illustration panel numbers.

- 1 Select the proper cuff size for the patient's limb circumference.
- 2 Apply cuff to limb **(1)** such that the side displaying the **Caution** symbol () is toward the patient and the arrow labeled **Arteria** is over the limb artery.
- 3 Wrap the cuff snugly around the patient's limb **(2 and 3)**. Ensure that the index edge falls within the range indicated by the range arrow **(4)**.
- 4 Secure the connector(s) on the cuff to the air hose connector to prevent leaking.